

# Quick Guide

## Perform test

Disclaimer, all patient data is randomly generated for demonstration purposes.

### Overview

The screenshot shows the OtoAccess software interface. Key components are labeled with arrows:

- Patient list:** A table on the left with columns for Patient ID, First name, and Last name. The patient 'Demo Interaco' is selected.
- Toolbar:** A top bar with icons for various modules like Affinity Suite, Micro-otological Vir..., EP, Titan Suite, Diagnostic SU, and Viot Suite.
- Preview:** A central area showing two graphs for 'Tymp 226 Hz Right' and 'Tymp 226 Hz Left', along with a table of test results for 'REFLEX GROWTH' and 'DECAY 1000 Hz'.
- Open report:** A button in the top right corner.
- Session list:** A list on the right showing sessions for the selected patient, including 'VisualEyes 525', 'ANSI 3.22 (2003)', and 'IMP440 - Acoustic Reflex Testing'.
- Language options:** A button at the bottom right.
- Help:** A button at the bottom right.

### Prerequisites

- Compatible equipment software installed, is visible in toolbar
- Patient created, see Quick guide create/delete/edit patient

### Perform tests

1. Choose patient in the patient list.
  - a. If there are existing sessions on the patient, these will be visible in the Session list.
  - b. Choosing sessions will show session preview if available and you can open the test report if available.
  - c. Double-clicking the session will open the session in the equipment software.
2. Click on the software tool that you want to use in the Toolbar. Perform tests on the selected patient. When you save a session it will automatically be saved to OtoAccess® Database.

